



Launch Tidepool Uploader

1. Open the **Chrome** web browser
2. In address bar, type **chrome://apps**
3. Click the **Tidepool Uploader** icon
4. Log in with **Tidepool** credentials



Find or create patient account

1. Search for **patient name**
2. Select patient and click **Next**
3. If none found, click **Add New**
4. If patient wants home access, enter **patient email**



Upload data

1. Select patient **devices**
2. Choose **time zone** and check that the **device time** corresponds.
3. Plug in device and click **Upload**
5. See data at **blip.tidepool.org**

Questions? We can help! Visit
support.tidepool.org

